

BIOPARAFREE



CODE	PACKAGING
08-1750	100 ml
08-1750-X4	4 x 100 ml



Manufacturer: Bio-Optica Milano S.p.A.

Date of emission: 22/06/2020

Rev. 02

PRODUCT DETAILS

Application: Microtomes and workbenches cleaning. It used for the removal of the residual paraffin. The product dissolves the paraffin residues for an easy removal.

Method: Atomize on the cleaning area, after 10 - 20 seconds, remove the residual paraffin.

Technical details

Specifications	Use	Paraffin removal		
	Density	848 kg/m ³ (20°C)		
	Viscosity	15 Cst (40°C)		
	Solubility	Insoluble in water and soluble in organic solvents.		
	Flammability	190°C		
Packaging	Primary packaging	White 100ml bottle in PEHD with atomizer.		
	Label	Wear, water, alcohol and solvents resistant label. Scratchproof ink resistant to water and alcohol.		
	Secondary packaging	Carton box		
Components	Mixture of mineral oils	CAS: -	CE: -	Index: -
Storage	Storage conditions	Store in a cool dry area, at room temperature. Keep away from sources of ignition. Keep from light. Keep the containers tightly closed.		
	Stability	After the first opening, the product is usable until the expiry date, if correctly stored.		
	Validity	2 years		
Warning	Functional characteristics	Mixture of mineral oils of high degree of purity according to FDA quality requirements. The product is odorless and colorless.		
	Product classification	The product must be used exclusively by specialized technical operators. Carefully read the information on the classification of dangerous substances on the label. Always refer to the safety data sheet where are available the information on the risks presented by the mixture, the precautionary measures during use, the measures first aid and the intervention in the event of accidental release. Do not use if the primary container is damaged.		
	Disposal	Hazardous preparation: observe all state and local environmental regulations regarding waste disposal.		

Date of emission: 22/06/2020

Rev. 02